


# HYDROGEN INHALATION MACHINE

## INSTRUCTION MANUAL INTRODUCTION, PARTS AND ACCESSORIES


Congratulations on your purchase of the H<sub>2</sub> Hydrogen Inhalation Machine! You have just acquired the world's most advanced and efficient technology for the production of Molecular Hydrogen and Oxygen mixed gas on-demand, in a form that conveniently sits on a countertop and is easily portable.

Each unit is hand-made in the U.S.A. with the highest quality materials and to exacting standards for a long life with very little maintenance. Your machine comes completely assembled and ready for use. It has been thoroughly tested prior to shipment, so there may be moisture in the Molecular Hydrogen generator. **This is normal.** Simply add distilled water!

### PARTS TOUR

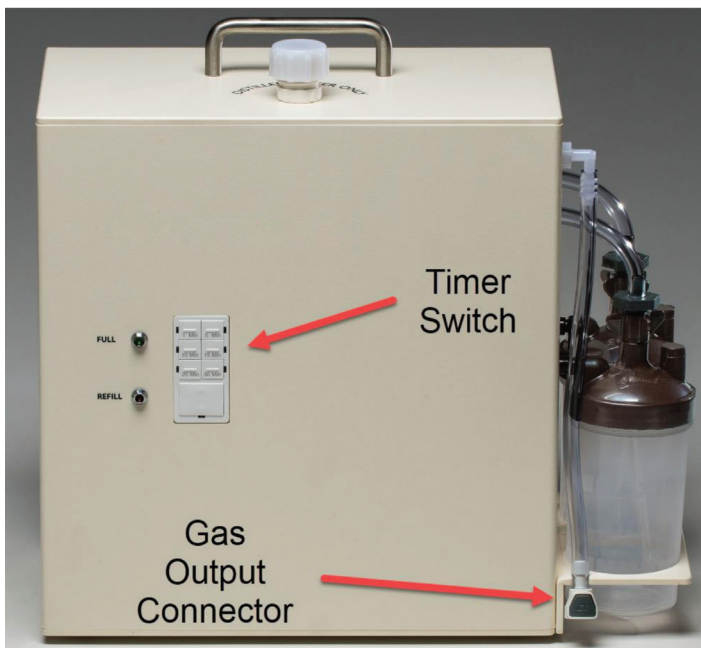


- Timer Switch: 1, 5, 10, 15, 20, 30 minute settings or Constant On
- Gas Output Connector: All accessories are connected here

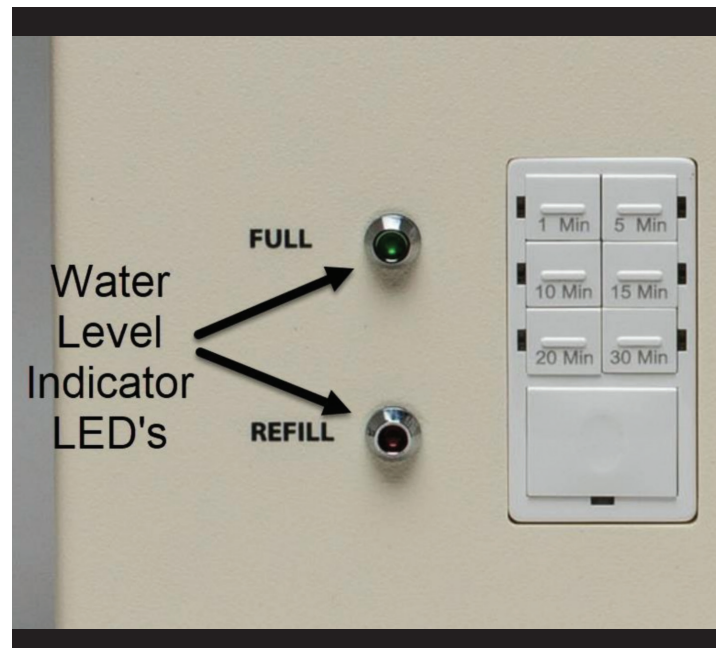


- Water Level Indicator LEDs
  - o FULL – Illuminated Green indicates full water level
  - o REFILL – Illuminated Red indicates it should be refilled

**Important Notes:** No LEDs illuminated means that the water level is above the Refill level, but below the Full level. This is where the unit will generally be most of the time. The unit will continue to run even with the Red "REFILL" LED illuminated, but should be refilled. Do not run empty. Due to the way the unit functions, the indicator lights may flicker and/or turn on/off.

**LED notice:** The water level rises slightly when the unit is started. This will cause the following behavior with the LED indicators. Both are normal! Ex: Green will be on, when full, and might turn off when the machine completes its run time. It might turn back on when the unit starts again. Red will be on after a timer cycle finishes, and if the unit is started, it might turn off again. You can refill the unit then.

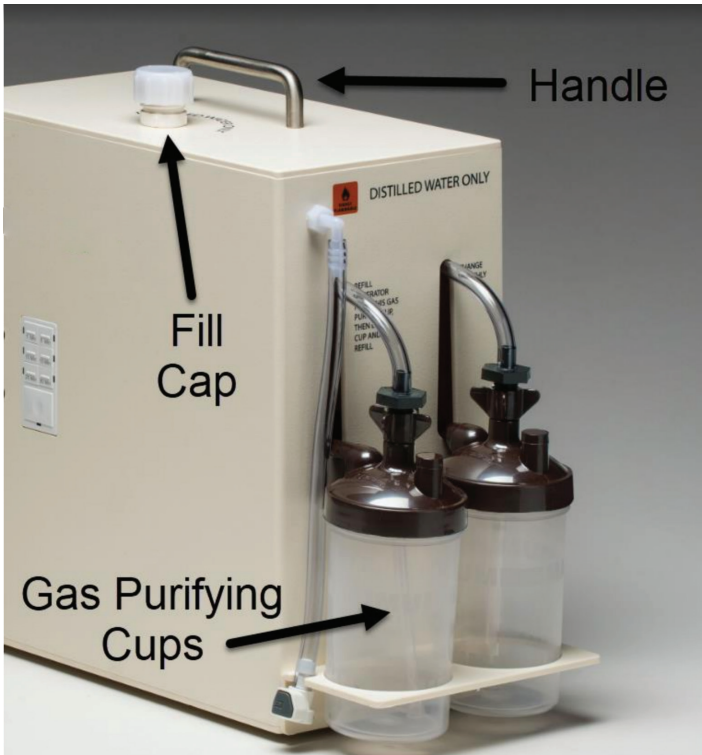

• Threaded Fill Cap • Gas Purifier Cups • Carrying Handle

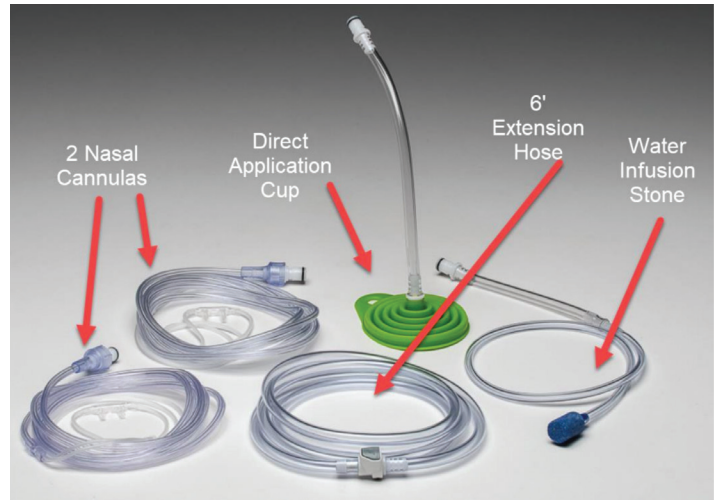

1. Two Nasal Cannulas (for Inhalation)
2. 6 foot Extension Hose
3. External Gas Applicator
4. Water Infusion Stone

Each accessory comes complete with a medical-grade connector that connects to the main gas output connector.

Visit [www.promolife.com/h2manual](http://www.promolife.com/h2manual) for safety, setup, usage, maintenance, cleaning and more.